

WARNING

- Remove the mask immediately if there is discomfort
- Do not use on Children (Under 12) or individuals who cannot remove the mask themselves.
- Do not cut or tear the mask. Do not use if damaged .If you come into contact with inner material brush off and rinse with water. In case of irritation seek medical advice.
- Whilst wearing the mask, do not apply pressure.
- Do not use if you have poor circulation, a reduced ability to feel sensation, diabetes, have had recent eye surgery or any medical condition which affects the eyes or the skin around the eyes.
- Keep out of reach of Children and pets.

CAUTIONS

- Use only in accordance with the instructions
- Do not bend or crush the eye mask before use
- Do not rub eyes during or immediately after use
- Do not microwave eye mask
- Do not allow mask to contact water or other liquids
- Do not use whilst wearing contact lenses, or immediately after applying eye drops
- Do not fall asleep whilst wearing the eye mask

STORAGE

Store in cool dry place avoiding direct sunlight